




Zee Tags Limited
Box 302-477
NORTH HARBOUR


Phone: 09 414 1790
Fax: 09 414 1794

HOW TO APPLY ZEE TAGS 1-PIECE EAR TAGS


FAST AND EASY TO APPLY

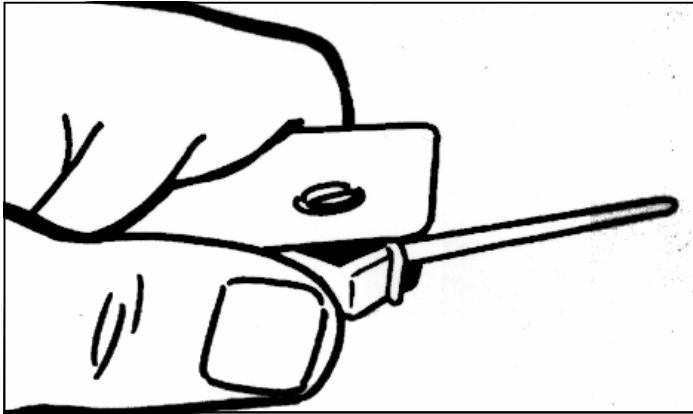

1 Flip applicator pin up to tag loading position, then hold as shown.

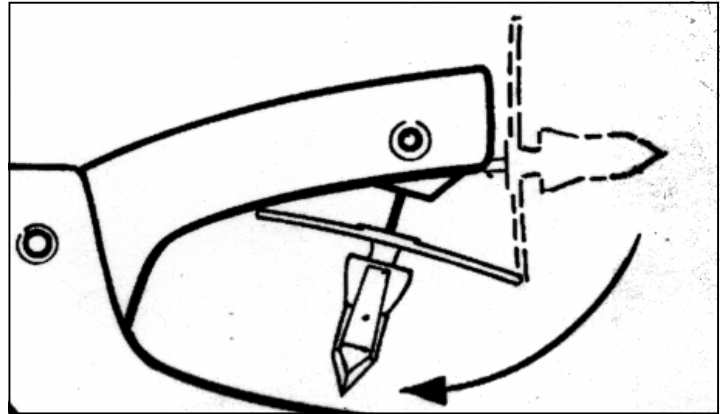

4 Flip applicator pin up to tag loading position, then hold as shown.

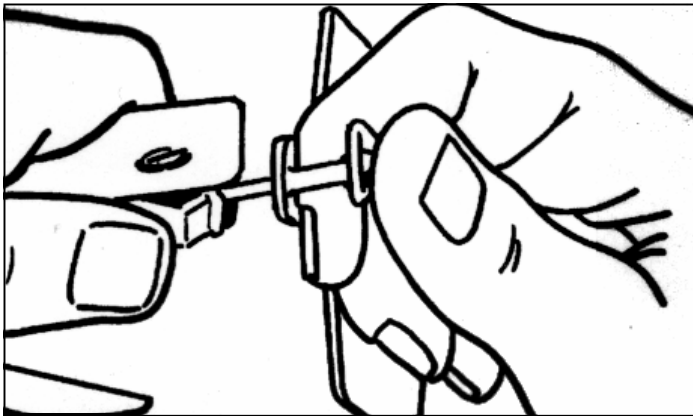


2 Insert the pin into the head while holding the tag as shown

Tag through the front of the ear only. To apply tag squeeze handles firmly and hold closed.
PLEASE NOTE
For animals with soft ears: Young Calves, goats and sheep. It is important to grasp tip of ear with a free hand and hold the ear taut to add support and ensure complete penetration of the tag through the ear.




5

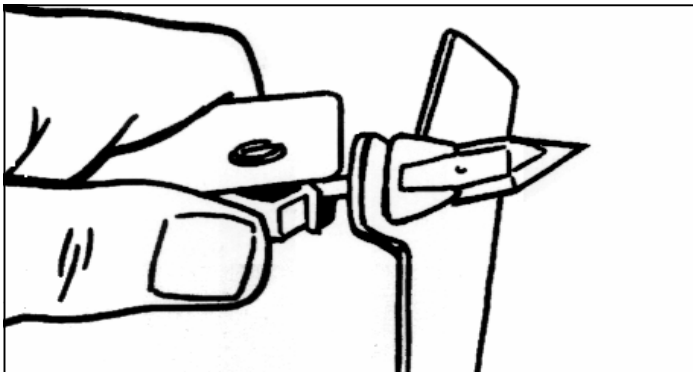

3 Straighten head of tag by lifting up with the forefinger & pushing down with the thumb, now push the tag tip completely onto pin.

To remove applicator: **WITHOUT OPENING HANDLES**, pull straight down from the animals ear in one smooth motion.

When the Zee Tag Applicator is pulled away from the ear, the pin automatically flips open. This prevents ripped ears and damage to the pin.

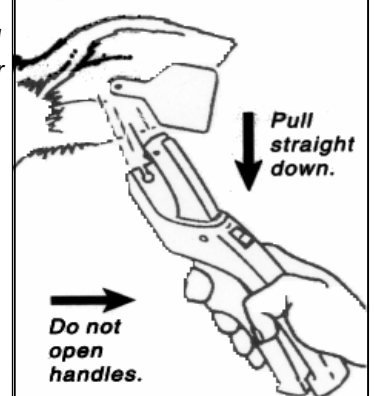

6


HELPFUL HINTS:

Need a spare pin?


There's one in the bottom handle of the applicator.

Keep your Zee Tag Applicator running smoothly.

Use a light lubricant on the applicator pin, such as WD40 or any silicon based product.

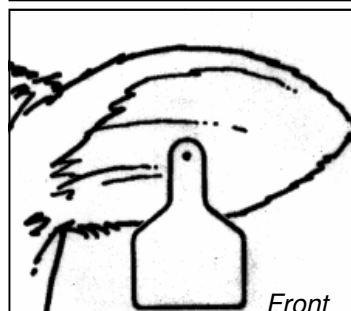

Front

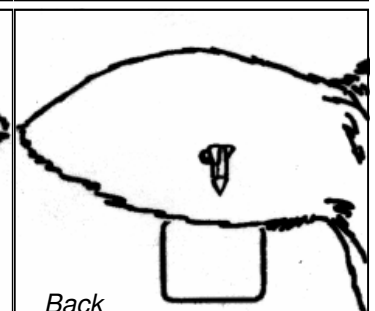

Back