





Zee Tags Limited
Box 302-477
NORTH HARBOUR
Phone 09 414 1790
Fax 09 414 1794

HOW TO APPLY ZEE TAGS 2-PIECE EAR TAGS


- Ensure you have a Zee Tag Two-Piece Tagger and that it is clean.
- Zee Tags are placed on Organisers that pair the male and female tag.
- Remove the tags from the packaging and arrange the tags in number sequence.
- Remove the complete set of tags with the same number. All these tags will go on one animal.
- Load the Tagger and follow the application instructions below:

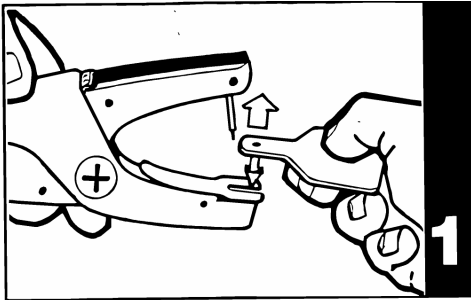

Push the male tag fully onto the tagger pin.

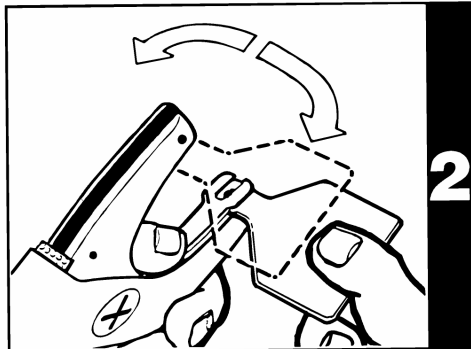


Depress the end of the female tag holder and place the female tag into position, making sure it is correctly aligned.

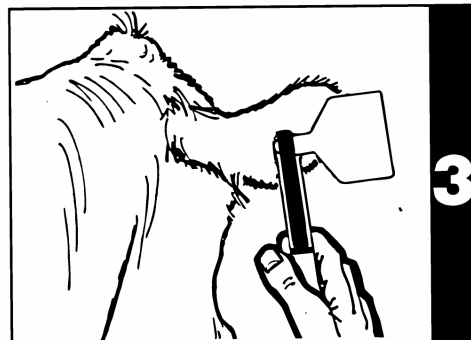

THE FEMALE TAG MUST ALWAYS GO ON THE FRONT OF THE EAR. The male tag is always applied from the back of the ear.

Position the tag between the two main cartilages, approximately half way between the animal's head and the ear tip. Squeeze the Tagger handles firmly closed. When the two tags have locked together the Tagger arm will flick back, (the pin may also flip out) and release the tag automatically. Then the Tagger can be freely withdrawn from the ear. This prevents ripped ears and damage to the pin.

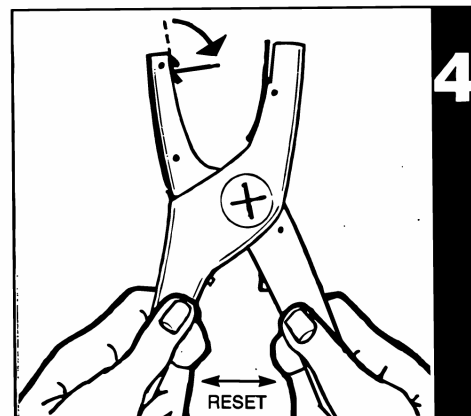

Reset the tagger and pin if necessary

HELPFUL HINTS

- Keep your Zee Tag applicator running smoothly. Use a light lubricant(or any silicone based product) on the tagger pin.